

Detartrine

Dental scaling / polishing



Properties

The **DETARTRINE** range is available as ready-to-use pastes designed for dental scaling and polishing.

- **DETARTRINE** is a pink paste, with a nice taste. Their mechanical abrasive action is due to the presence of silica, which makes it possible to complete the removal of dental tartar, after using an ultrasonic or hand scaler, without causing erosion of dental enamel.

The essential oils incorporated into **DETARTRINE** give a feeling of freshness, which is very much appreciated by the patients. In addition, they have a slight balsamic effect: they relieve the irritation that mechanical scaling might cause on sensitive gums.

Indications

- Finishing of scaled surfaces,
- Removal of food and tobacco stains,
- Fine polishing of fillings.

Instructions for use

Take the required amount of paste with a spatula and place it onto a polishing wheel.

Note: Mouth rinsing can be performed during or at the end of scaling.

Precautions for use

Possible allergic reaction to formaldehyde.

Presentations

45 g jar.
150 g jar.

For professional dental use only

Детартрин

Удаление зубного камня / полировка



Свойства

ДЕТАРТРИН представляет собой готовую к использованию пасту для удаления зубного камня и полировки поверхности зубов.

- **ДЕТАРТРИН** представляет собой розовую пасту с приятным вкусом. Благодаря входящему в его состав силикатам, он обладает механическим абразивным действием, которое обеспечивает полное удаление зубного налета после использования ультразвукового или ручного скалера, не вызывая повреждения зубной эмали.

Эфирные масла, входящие в состав **ДЕТАРТРИНА**, дают ощущение свежести, которое очень ценится пациентами. Помимо этого, эфирные масла обладают легким бальзамическим эффектом: они уменьшают раздражение десен, которое может возникнуть в результате механического снятия зубных отложений.

Показания

- Финишная обработка после удаления зубных отложений;
- Удаление пятен от пищи и табака;
- Тонкое шлифование пломб.

Способ применения

Взять на шпатель необходимое количество пасты и нанести ее на полировочный диск.

Примечание: полоскание рта может быть выполнено во время процедуры удаления налета или после нее.

Меры предосторожности

Возможен риск аллергии на формальдегид.

Формы выпуска

Баночка 45 г.
Баночка 150 г.

РУ № ФСЗ 2011/09767 от 24/05/2011

Только для профессионального применения в стоматологии

