32-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016 JAPAN  
Phone: +81 3 5817-8665 Fax: +81 3 5845-2965

Date: 25<sup>th</sup> of March 2025

«Approved by»

Hideki Kazama

President

Tokuyama Dental Corporation

INSTRUCTION FOR USE  
FOR THE MEDICAL DEVICE

Dental cement ESTECCEM II PLUS PASTE in syringes.

Version 2

## 1. Medical device name

### Dental cement ESTECER II PLUS PASTE in syringes

#### Versions:

1. **ESTECER II PLUS PASTE UNIVERSAL** consisting of:
  - Cement in double syringe (PASTE A (2.3 ml (4.7 g)) and PASTE B (2.3 ml (4.7g))), UNIVERSAL shade (universal).
  - Cement Mixing Tip – 5 pcs. In a package together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - MINI MIXING TIP S – 2 pcs. together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - NOZZLE FOR MINI MIXING TIP – 2 pcs. together with the main device or 40 pcs. in a separate package, -not more than 25 packages. (if necessary)
  - Instruction –leaflet.
  - Instruction for use (if necessary).
2. **ESTECER II PLUS PASTE CLEAR** consisting of:
  - Cement in double syringe (PASTE A (2.3 ml (4.7 g)) and PASTE B (2.3 ml (4.7g))), CLEAR shade (clear).
  - Cement Mixing Tip – 5 pcs. In a package together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - MINI MIXING TIP S – 2 pcs. together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - NOZZLE FOR MINI MIXING TIP – 2 pcs. together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - Instruction –leaflet.
  - Instruction for use – (if necessary).
3. **ESTECER II PLUS PASTE BROWN** consisting of:
  - Cement in double syringe (PASTE A (2.3 ml (4.7 g)) and PASTE B (2.3 ml (4.7g))), BROWN shade (brown).
  - Cement Mixing Tip – 5 pcs. In a package together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - MINI MIXING TIP S – 2 pcs. together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - NOZZLE FOR MINI MIXING TIP – 2 pcs. together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - Instruction –leaflet.
  - Instruction for use (if necessary).
4. **ESTECER II PLUS PASTE WHITE OPAQUE** consisting of:
  - Cement in double syringe (PASTE A (2.3 ml (4.7 g)) and PASTE B (2.3 ml (4.7g))), WHITE OPAQUE shade (white opaque).
  - Cement Mixing Tip – 5 pcs. In a package together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - MINI MIXING TIP S – 2 pcs. together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - NOZZLE FOR MINI MIXING TIP – 2 pcs. together with the main device or 40 pcs. in a separate package, -not more than 25 packages (if necessary).
  - Instruction –leaflet.
  - Instruction for use (if necessary).

Hereinafter referred to as material, device, paste, cement, ESTEC EM II PLUS PASTE, PASTE A, PASTE B.

## **2. Information about the manufacturer, designer and authorized representative**

### **Designer**

Tokuyama Dental Corporation  
38-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016, Japan  
Tel.: +81-3-3835-7201  
URL: <http://www.tokuyama-dental.com>

### **Manufacturer**

Tokuyama Dental Corporation  
38-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016, Japan  
Tel.: +81-3-3835-7201  
URL: <http://www.tokuyama-dental.com>

### **Manufacturing site**

Tokuyama Dental Corporation Kashima Factory,  
26, Sunayama, Kamisu-shi, Ibaraki 314-0255, Japan

### **Authorized representative of the manufacturer in the Russian Federation**

PROTECO JSC  
Legal address: 192102, St. Petersburg, 57 Salova str., bldg. 1, lit. I.  
Tel.: +7 (812) 635-88-90; Email: [info@protecodent.ru](mailto:info@protecodent.ru)

## **3. Classification**

ISO 4049 classification: ESTEC EM II PLUS (PASTE A, PASTE B)  
Type 2: all other polymer based restorative materials, and luting materials.  
Class 3: materials that are cured by the application of external energy and also have a self-curing mechanism present ("dual-cure" materials).

## **4. Intended use**

This device is intended for cementation of dental prosthetic materials such as metals, ceramics and composite materials to tooth structure or in case of their repair.

## **5. Potential consumer, scope of application**

Potential consumers of a medical device: dentists.

Scope of application: dentistry.

## **6. Indications, contraindications and possible side effects associated with the use of the medical device as directed**

### Indications

Cementation of crowns, bridges, inlays, and onlays made of glass/oxide ceramics (porcelain, zirconia and alumina), metals/alloys (precious and non-precious) and resin materials including inorganic filler (composite materials).

---

INSTRUCTION FOR USE FOR THE MEDICAL DEVICE

Dental cement ESTEC EM II PLUS PASTE in syringes.

Page 3 of 13

- Repair of fractured porcelain fused to metal crowns and all ceramic restorations.
- Cementation of veneers.
- Cementation of adhesion bridges.
- Cementation of metal or resin cores, metal or glass-fiber posts.

#### Contraindications:

This device contains methacrylic monomers, peroxide. DO NOT use this device for patients allergic to or hypersensitive to methacrylic monomers, peroxide.

Possible side effects: allergic reactions to or hypersensitive to methacrylic and similar monomers, peroxide or any other ingredients.

### **7. Description of the device, principle of operation of the medical device**

Cement ESTECCEM II PLUS PASTE is a dual-cure (light and/or self-cure) radiopaque, adhesive resin cement.

Cement ESTECCEM II PLUS PASTE is excellent in handling, esthetic and adhesive properties to hard tissues of teeth and all prosthetic materials (co-used with Tokuyama Universal Bond II).

By curing the resin of PASTE (A and B) applied to an adhesion interface between the tooth and the prosthesis, those adherends bond by the cured resin.

Cement ESTECCEM II PLUS PASTE (A/B) is dispensed in an automix syringe and is available in various shades (UNIVERSAL, CLEAR, BROWN, WHITE OPAQUE).

The PASTE contains Bisphenol A di(2-hydroxy propoxy) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate (TEGDMA), Bisphenol A polyethoxy methacrylate-MPEPP, Peroxide, Camphorquinone and Silica-Zirconia Filler (Filler Load 74 wt%).

Other components, included in the composition of the device: Cement Mixing Tip, MINI MIXING TIP S, NOZZLE FOR MINI MIXING TIP, instruction leaflet, instruction for use.

The material needs to be used with Tokuyama Universal Bond II in kit (Registration certificate No. RZN 2023/19792 dd. 15.03.2023) Tokuyama Universal Bond II promotes the adhesion of ESTECCEM II PLUS PASTE to tooth structure. Tokuyama Universal Bond II also enhances the adhesion of ESTECCEM II PLUS PASTE to all prosthetic materials.

For information on contraindications, precautions, storage, handling, etc. of Tokuyama Universal Bond II in kit, please see the Tokuyama Universal Bond II Instruction for use.

### **8. Precautions**

- Use Tokuyama Universal Bond II as primer for ESTECCEM II PLUS PASTE. DO NOT use other primers or adhesives as primer for adhesive surfaces.
  - DO NOT USE ESTECCEM II PLUS PASTE for any purpose other than those specified in this manual. Use ESTECCEM II PLUS PASTE only as described below.
  - ESTECCEM II PLUS PASTE is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.
  - DO NOT USE the ESTECCEM II PLUS PASTE if the safety seals are broken or appear to have been tampered with.
  - If ESTECCEM II PLUS PASTE causes an allergic reaction or oversensitivity, discontinue its use immediately.
  - Use examination gloves (plastic, vinyl or latex) at all times when handling ESTECCEM II PLUS PASTE to avoid the possibility of allergic reactions from methacrylic monomers.
- Note: Certain substances/materials may penetrate through examination gloves. If ESTECCEM

- II PLUS PASTE (PASTE A and B) comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- Avoid contact of ESTECER II PLUS PASTE with eyes, mucousal membrane, skin and clothing.
    - If ESTECER II PLUS PASTE comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
    - If ESTECER II PLUS PASTE comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration is seated. The affected area may turn white due to coagulation of the protein, but this white discolouration disappears on its own within 24 hours. Instruct the patient to seek immediate medical attention if the spot does not disappear within 24 hours.
    - If ESTECER II PLUS PASTE comes into contact with the skin or clothing, immediately saturate the area with alcohol-soaked cotton swab or gauze.
    - Instruct the patient to rinse his mouth immediately after treatment.
  - ESTECER II PLUS PASTE should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
  - To avoid the unintentional ingestion of ESTECER II PLUS PASTE, do not leave it unsupervised within the reach of patients and children.
  - DO NOT use ESTECER II PLUS PASTE directly to the cavity being in close proximity to the pulp. Pulp protection with glass ionomer lining or calcium hydroxide is recommended. DO NOT use EUGENOL BASED MATERIALS to protect the pulp as these materials will inhibit the curing of Tokuyama Universal Bond II.
  - Wear protective shields, goggles or visors at all times when working with the light curing unit.
  - Be aware that when a restoration is chipped from misaligned occlusion or bruxism (clenching, grinding or tapping), the repaired restoration may chip again.
  - DO NOT mix-ESTECER II PLUS PASTE with other brands of paste.
  - ESTECER II PLUS PASTE should be brought to room temperature before using. After removing from refrigerator, allow to sit for 20 minutes or until it reaches room temperature. Return ESTECER II PLUS PASTE to the refrigerator after using.
  - DO NOT use after the expiration date on the syringe/package.
  - Use the Cement Mixing Tip or MINI MIXING TIP S included in this device when extruding the mixed paste.
  - Use a clean instrument or explorer to apply or remove the paste.
  - To avoid unexpected hardening of ESTECER II PLUS PASTE, after each use remove the mixing-tip from the automix syringe. Using a clean cloth or tissue wipe the excess paste from the tip of syringe prior to recapping the syringe using the cap provided with the syringe at the time of purchase.

## 9. Application procedure

### 1. Tooth preparation

Remove contamination from the tooth surface such as plaque, temporary cement residue, oil from materials to test crown fit, oil mist from a hand-piece, saliva, blood and exudates fluids by brushing with flour pumice, ultrasonic scaling, or cleaning the surface of the tooth with alcohol (not less than 70%). Thoroughly rinse and dry the surface.

If the adherent surface involves uncut enamel, apply etching agent to the uncut enamel. See section "Selective-enamel-etch technique".

DO NOT desiccate the vital tooth. Desiccation can lead to post-operative sensitivity.

### 2. Etching of the tooth structure (optional)

#### a) Selective-enamel-etch technique

Apply phosphoric acid gel only to the enamel surrounding the margin of prepared cavity and leave phosphoric acid gel in place for 10-15 seconds. Rinse the etched surface thoroughly (at least 15 seconds) with water, and then dry with mild air.

b) Total etch technique

Apply phosphoric acid gel onto the prepared enamel first, and then the dentin. The etchant should be left to react on the enamel and dentin for 10-15 seconds. Rinse thoroughly (at least 15 seconds) with water and then dry with mild air or with cotton pellets; do not overdry.

### 3. Restoration Preparation

Follow the procedure shown below, depending on the material to be restored.

1) Ceramics, other than Porcelain, and Composite materials

Roughen the interior of the restoration by sandblasting (0.1 to 0.2 MPa), air abrasion, or grinding with a diamond bur using a slow speed to prepare the surface for adhesion. Thoroughly rinse and dry the surface.

2) Porcelain

DO NOT roughen the surface. Prepare the interior of the restoration, providing a clean and fresh surface for adhesion. Thoroughly rinse and dry the surface.

3) Metal restorations

Roughen the area by sandblasting (0.3 to 0.5 MPa), air abrasion or grinding with a diamond bur using a slow speed to prepare the surface for adhesion. Thoroughly rinse and dry the surface.

- When using the sandblaster or air abrasion, use at least 30-to-50-micron grid of alumina powder.
- If unable to roughen or prepare the adhesive surface, etch the area for adhesion with phosphoric acid then thoroughly rinse and dry.
- The substances listed below inhibit adhesion and must be removed from the adhesive surface by mechanical cleaning or alcohol (not less than 70%) soaking. Clean the restoration after adjustment is complete.
  - 1) Gypsum and investment.
  - 2) Silicone oil from fit check material.
  - 3) Oil mist from headpiece.
  - 4) Plaque, tartar, saliva, blood and exudate fluids.
  - 5) Temporary restoration and temporary cement.
- If necessary, protect the surface not being bonded with metal separator, paraffin film, vaseline, etc.

### 4. Dispensing of Tokuyama Universal Bond II

Allow the Tokuyama Universal Bond II to reach room temperature prior to use. Open the bottle by removing the cap and dispense one drop each of Tokuyama Universal Bond II A and B into mixing well or disposable mixing well. Mix thoroughly with a disposable applicator until the mixed bonding agent turns green.

- Dispense Tokuyama Universal Bond II B IMMEDIATELY after dispensing Tokuyama Universal Bond II A. Tokuyama Universal Bond II A may turn pink due to solvent volatilization if allowed to sit too long. If Bond A turns pink, dispose of it and re-dispense fresh Tokuyama Universal Bond II A and B.
- Complete the application within 1 minute of dispensing when using the mixing well since Tokuyama Universal Bond II contains volatile solvents. After mixing, the color of Tokuyama Universal Bond II changes gradually based on the reaction of the catalyst of Tokuyama Universal Bond II, this is normal and does not indicate problem with Tokuyama Universal Bond II.
- Complete the application within 3 minutes of dispensing when using the disposable mixing well.

- DO NOT touch the nozzle to the mixing well while dispensing. DO NOT interchange the bottle caps of Tokuyama Universal Bond II A and B. If Tokuyama Universal Bond II A and B come into contact while dispensing, Tokuyama Universal Bond II may cure in the nozzle.
- Wipe off excess Tokuyama Universal Bond II from the tip of the tip of the nozzle and the inside of the cap prior to closing.
- Immediately after dispensing, tightly replace the bottle caps of Tokuyama Universal Bond II A and B.
- Do not over tighten the bottle caps.

## **5. Restoration Pretreatment with the Tokuyama Universal Bond II**

### **1) Application**

Using the disposable applicator, apply the mixed Tokuyama Universal Bond II on the surface to be bonded.

- 1) Ensure Tokuyama Universal Bond II covers all surfaces, where ESTECER II PLUS PASTE will be applied.
- 2) DO NOT over apply. One application of Tokuyama Universal Bond II is sufficient. Too thick of a layer may compromise the fit of the restoration.
- 3) DO NOT allow saliva, blood, oil, water, etc. to come in contact with the surface to be bonded. If the treated surface is contaminated, thoroughly rinse with water, dry and re-apply fresh Tokuyama Universal Bond II.
- 4) DO NOT wait too long after the application of Tokuyama Universal Bond II. Air dry within 30 seconds after application, to ensure the appropriate film thickness.

### **2) Air dry**

Using an oil-free air/water syringe, apply mild air to the surface. Use a vacuum aspirator to prevent spatter of the Tokuyama Universal Bond II.

- Be sure to dry the surface after Tokuyama Universal Bond II is applied. Incomplete drying may result in poor adhesion.

## **6. Tooth (Adherent surface) pretreatment with Tokuyama Universal Bond II**

### **1) Application**

Using the disposable applicator, apply the mixed Tokuyama Universal Bond II on the surface to be bonded.

- Ensure Tokuyama Universal Bond II covers all surfaces, where ESTECER II PLUS PASTE will be applied. Even if the adherent surface includes materials other than tooth structure such as ceramics, porcelain, composite, and/or metal, no extra primers are needed.
  - Dispensed Tokuyama Universal Bond II can be used for both the restoration and tooth pretreatment if used within 1 minute using the mixing well or 3 minutes if using the disposable mixing well.
  - If the time after dispensing is over 1 minute (mixing well), or over 3 minutes (disposable mixing well), DO NOT use dispensed Tokuyama Universal Bond II due to the possibility of the poor coating properties or insufficient adhesion. If the time after dispensing exceeds the limit (1 minute or 3 minutes), dispense again one drop each of Tokuyama Universal Bond II A and B into the mixing well or disposable mixing well and mix thoroughly with a disposable applicator.
- If the Tokuyama Universal Bond II pools on the cavity floor or a cavosurface angle and is too thick to air thin, remove the excess with a paper point or with a new disposable applicator.
- Be careful not to allow excess Tokuyama Universal Bond II to flow into the subgingival. If Tokuyama Universal Bond II flows into the subgingival, thoroughly rinse and dry the area.

- DO NOT wait too long after the application of Tokuyama Universal Bond II; this will lead to the possibility of the restoration not seating properly with cementation. Air dry within 30 seconds after application, to ensure the appropriate film thickness.

## 2) Air Dry

Using an oil-free air/water syringe, apply weak air continuously to the Tokuyama Universal Bond II surface until the runny Tokuyama Universal Bond II stays in the same position without any movement. Finish by using mild air to the surface. Use a vacuum aspirator to prevent spatter of the Tokuyama Universal Bond II.

- If saliva, blood, or other fluids contaminate the applied Tokuyama Universal Bond II, thoroughly rinse with water, dry and re-apply fresh Tokuyama Universal Bond II.
- DO NOT rinse the applied Tokuyama Universal Bond II with water except in unintentional contamination.
- If accidental spattering occurs, it may cause the tissue to whiten or possible allergic reaction.

## 7. Cementation and final bonding

### 1) Preparing the syringe and mixing tip

Before each use, remove the cap from automix syringe, and squeeze out a small quantity of PASTE A and B, making sure equal amounts are being dispensed through the two outlets of the syringe.

- How to remove the cap: turn the cap 1/4 of a turn counterclockwise to align the projections on the cap with the grooves in the syringe. Holding the base of the cap, remove the cap by twisting and pressing downward.
- DO NOT allow PASTE A to come into contact with PASTE B, as there is a possibility of hardening of the paste at the tip of the syringe. Wipe the excess paste from the tip of the syringe with a clean cloth or tissue before using a new mixing tip.
- If equal amounts of PASTE A and B are not dispensed, there is a possibility of poor polymerization. Ensure equal amounts are being dispensed through the two outlets of the syringe prior to attaching a mixing tip.
- How to attach the mixing tip: align the projections on the adapter with the grooves in the syringe and turn the mixing tip 1/4 of a turn clockwise to lock it in place.
- After use, remove the mixing tip from the automix syringe after use and replace the cap of the automix syringe.

### 2) Cementation

#### a) Cementation of Restoration

Attach a Cement Mixing Tip to the syringe, squeeze PASTE (A and B) from the syringe and apply the mixed paste to the inner surface of the restoration. Place the restoration on the tooth with firm pressure. Any excess paste at the margin can be removed with the brush tip.

- DO NOT apply the paste directly to the prepared tooth or the cavity treated with Tokuyama Universal Bond II to avoid incorrect seating of the restoration.
- To keep the appropriate film thickness, seat the restoration as soon as possible after the application of the paste (within 1 minute after dispensing), if not there is a possibility that the restoration will not seat properly with cementation.

#### b) Placing the core or post

Attach a Mini Mixing Tip S and a Nozzle for Mini Mixing Tip to the syringe for application in the root canal. Apply the mixed paste over the entire adherent surface of the core or post, or the entire tooth surface within the cavity. Place the core or post quickly into the cavity, slightly vibrating it to prevent air bubbles from entering the root canals.

- If the paste is applied directly into the cavity, place the core or post into the cavity within 30 seconds after application of the paste.
- DO NOT use Lentulo spirals to insert the paste in the root canal, as this can excessively accelerate setting.

## 8. Removal of Excess paste

There are two methods for removal of excess paste.

### a) When using the light-cure method

Once the restoration is seated, light-cure the excess paste with a dental curing light for 2 to 4 seconds. The gel state of the excess paste can be easily removed.

- Excess light curing will lead to difficult removal of the excess paste.
- Tokuyama Universal Bond II layer may appear fluorescent under the dental curing light irradiation.

### b) When using the self-cure method

The gel state of the excess paste can be easily removed within 1 to 3.5 minutes after seating.

- Waiting more than 3.5 minutes after seating will lead to difficult removal of the excess paste.
- The setting time of the paste may change (approximately 20 seconds at the maximum) depending on the oral temperature in each clinical case.

## 9. Final Adjustment

### 1) Final paste hardening

- a) In case of translucent restoration materials such as ceramics or composite materials, Irradiate the light to the the applied paste on the seated restoration for 20 seconds or more. When the adherent area is too large to be completed by a single light-cure, multiple cures are recommended.

Confirm that the light-curing unit has sufficient intensity ( $>500\text{mW/cm}^2$ ) before using.

Note that using a cracked light guide will lower the intensity.

- b) In case of non translucent restoration materials such as metal, Light-cure along margins for 20 seconds or more, then allow the paste to set for 8 minutes, with the patient applying normal occlusal pressure to the restoration.

### 2) Finishing

Check patient's bite, if necessary. Re-polish the restoration with the material of choice.

**IMPORTANT NOTE:** The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure that this product is suitable for an appropriate application before use.

Specifications are subject to change without notice. When the product specifications changes, the instructions and precautions may also change.

## 10. Specification of ESTECER II PLUS PASTE

| Test item         | Specification                                                                                                                                                               |
|-------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Film thickness    | Maximum 50 $\mu\text{m}$                                                                                                                                                    |
| Working time      | The material shall be capable of forming a thin layer at 60 seconds after completion of mixing; during its formation there shall be no detectable change in its homogeneity |
| Setting time      | Maximum 10 minutes                                                                                                                                                          |
| Flexural strength | Minimum 50 MPa                                                                                                                                                              |
| Water sorption    | Maximum 40 $\mu\text{g/mm}^2$                                                                                                                                               |
| Solubility        | Maximum 7.5 $\mu\text{g/mm}^2$                                                                                                                                              |
| Color stability   | No more than a slight color change shall be observed after 7 days water sorption and after 24 hours of light irradiation                                                    |
| Radio-opacity     | The radio-opacity of the set material shall be equal to or greater than that of the same thickness of aluminum                                                              |
| Appearance        | Homologous mixed paste                                                                                                                                                      |

### 11. Material compatibility

As described in the introduction, this device is a dental cement, so this device would contact with tooth structure and various materials used as prostheses in dentistry. This device being used with adhesive (Tokuyama Universal Bond II in kit, Registration certificate No. RZN 2023/19792 dd. 15.03.2023) has enough bond strength to enamel and dentin with or without phosphoric acid etching and to prosthesis materials such as metals/alloys, glass/oxide ceramics and composite materials.

Some materials and medicaments (hemostatic material) inhibit setting/adhesion of Tokuyama Universal Bond II. DO NOT use products that contain:

- eugenol,
- aluminum chloride,
- ferric sulfate,
- calcium sulfate,
- aluminum sulfate,
- diammine silver fluoride [molecular formula:  $\text{Ag}(\text{NH}_3)_2\text{F}$ ]

Tokuyama Universal Bond II does not adhere to tooth structure immediately after applications of materials that contain:

- hydrogen peroxide (oxydol),
- sodium hypochlorite

We advise avoiding the use of these materials on the day of cementation, however these materials can be used for root canal treatment as the inhibition caused by these materials usually evaporates within five days.

### 12. Devices incorporating medicinal products or substances absorbed by or dispersed in human body

Not applicable. This device is not a device incorporating a substance considered to be medicinal product, and is not a device that is composed of substances that are absorbed by or locally dispersed in the human body.

### 13. Devices incorporating materials of biological origin

Not applicable. This device is not a device incorporating a material of biological origin.

### 14. List of standards that the medical device complies with

The standards and specifications that apply to "Dental cement ESTECER II PLUS PASTE in syringes" to ensure that it meets the General Safety Performance Requirements (GSPR) of the MDR and the Essential Requirements of the MDD.

#### List of standards:

EN ISO 10993-1, EN ISO 10993-18, ISO 7405, EN ISO 13485, EN ISO 14971, EN 1041, EN 1641, EN ISO 15223-1, ISO/TS 11405, ISO 4049.

## 15. Scope of supply.

| Device name                                                                                                          | Quantity                                                                                          |
|----------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------|
| <b>I. ESTECCEM II PLUS PASTE UNIVERSAL</b>                                                                           |                                                                                                   |
| 1. Cement in double syringe (PASTE A (2.3 ml (4.7 g) and PASTE B (2.3 ml (4.7g)), UNIVERSAL shade (universal).       | 1 pc.                                                                                             |
| 2. Cement Mixing Tip                                                                                                 | 5 pcs. In a package together with the main device or 40 pcs. in a separate package (if necessary) |
| 3. MINI MIXING TIP S                                                                                                 | 2 pcs. together with the main device or 40 pcs. in a separate package (if necessary)              |
| 4. NOZZLE FOR MINI MIXING TIP                                                                                        | 2 pcs. together with the main device or 40 pcs. in a separate package (if necessary)              |
| 5. Instruction –leaflet. *                                                                                           | 1 pcs.                                                                                            |
| 6. Instruction for use (if necessary) **                                                                             | 1 pcs.                                                                                            |
| <b>II. ESTECCEM II PLUS PASTE CLEAR</b>                                                                              |                                                                                                   |
| 1. Cement in double syringe (PASTE A (2.3 ml (4.7 g) and PASTE B (2.3 ml (4.7g)), CLEAR shade (clear).               | 1 pcs.                                                                                            |
| 2. Cement Mixing Tip                                                                                                 | 5 pcs. In a package together with the main device or 40 pcs. in a separate package (if necessary) |
| 3. MINI MIXING TIP S                                                                                                 | 2 pcs. together with the main device or 40 pcs. in a separate package (if necessary)              |
| 4. NOZZLE FOR MINI MIXING TIP                                                                                        | 2 pcs. together with the main device or 40 pcs. in a separate package (if necessary)              |
| 5. Instruction –leaflet. *                                                                                           | 1 pc.                                                                                             |
| 6. Instruction for use (if necessary) **                                                                             | 1 pc.                                                                                             |
| <b>III. ESTECCEM II PLUS PASTE BROWN</b>                                                                             |                                                                                                   |
| 1. Cement in double syringe (PASTE A (2.3 ml (4.7 g) and PASTE B (2.3 ml (4.7g)), BROWN shade (brown).               | 1 pc.                                                                                             |
| 2. Cement Mixing Tip                                                                                                 | 5 pcs. In a package together with the main device or 40 pcs. in a separate package (if necessary) |
| 3. MINI MIXING TIP S                                                                                                 | 2 pcs. together with the main device or 40 pcs. in a separate package (if necessary)              |
| 4. NOZZLE FOR MINI MIXING TIP                                                                                        | 2 pcs. together with the main device or 40 pcs. in a separate package (if necessary)              |
| 5. Instruction –leaflet. *                                                                                           | 1 pcs.                                                                                            |
| 6. Instruction for use (if necessary) **                                                                             | 1 pcs.                                                                                            |
| <b>IV. ESTECCEM II PLUS PASTE WHITE OPAQUE</b>                                                                       |                                                                                                   |
| 1. Cement in double syringe (PASTE A (2.3 ml (4.7 g) and PASTE B (2.3 ml (4.7g)), WHITE OPAQUE shade (white opaque). | 1 pcs.                                                                                            |
| 2. Cement Mixing Tip                                                                                                 | 5 pcs. In a package together with the main device or 40 pcs. in a separate package (if necessary) |
| 3. MINI MIXING TIP S                                                                                                 | 2 pcs. together with the main device or 40 pcs. in a separate package (if necessary)              |
| 4. NOZZLE FOR MINI MIXING TIP                                                                                        | 2 pcs. together with the main device or 40 pcs. in a separate package (if necessary)              |
| 5. Instruction –leaflet. *                                                                                           | 1 pcs.                                                                                            |
| 6. Instruction for use (if necessary) **                                                                             | 1 pcs.                                                                                            |

\* – The instruction leaflet for each package of each version of medical devices contains abbreviated information in accordance with the requirements of Order 11N dated 19.01.2017, Section II., paragraph 8. of the Ministry of Health of the Russian Federation.

\*\* – Provided by the distributor or seller if necessary. It is also presented electronically on the website of the authorized representative <https://protecodent.ru>.

## 16. Interpretation of symbols used in the labeling.

### Symbols and description.

| Symbol                                                                            | Description                  | Symbol                                                                            | Description                 |
|-----------------------------------------------------------------------------------|------------------------------|-----------------------------------------------------------------------------------|-----------------------------|
|  | Manufacturer                 |  | Lot number                  |
|  | Catalog number               |  | Expiration date             |
|  | Consult instructions for use |  | Prohibition of repeated use |
|  | Temperature limit            |                                                                                   |                             |

## 17. Requirements for the maintenance and repair of a medical device

Devices do not need maintenance and repair.

## 18. Sterilization, cleaning and disinfection

The medical device is supplied non-sterile and is not intended to be sterilized before use.

Cement Mixing Tip, MINI MIXING TIP S, NOZZLE FOR MINI MIXING TIP are single-use accessories and should not be reused according to the instructions for use.

## 19. Conditions of transportation, storage and operation

Transportation of devices packed in shipping containers should be carried out by all types of closed vehicles in compliance with the precautions indicated on the shipping container and in accordance with the rules for the carriage of goods in force for this type of transport.

Shipping conditions:

- Temperature – from 0 to +10 °C;
- Relative humidity: 10 – 100 %;
- Atmospheric pressure: 800 ~ 1.060 hPa.

Store in the refrigerator. Keep away from heat, direct sunlight, sparks and open flames.

Storage conditions:

- Temperature – from 0 to +10 °C;
- Relative humidity: 10 – 70%;
- Atmospheric pressure: 900 ~ 1.060 hPa.

Indoor conditions when using a medical device

- Temperature from +18 °C to +30 °C

## 20. Shelf life

Shelf life is 2 years.

## 21. Procedure for the disposal and destruction of a medical device

The medical device "Dental cement ESTECER II PLUS PASTE in syringes" depending on the degree of their epidemiological, toxicological and radiation hazard, is classified as epidemiologically safe waste, similar in composition to solid domestic waste (SDW). However, it is recommended to dispose of them separately as industrial waste through specialized organizations licensed for this type of activity.

The following should be done for the safe disposal of product residues: remove unused dental cement ESTECER II PLUS PASTE (A and B) from the syringe, mix and allow to harden;

The devices should be collected and handed over to a specially licensed medical waste disposal organization. Waste transportation from institutions to the place of disposal is carried out by specialized companies.

Collection of non-hazardous waste is carried out in reusable containers or disposable bags (with the exception of bags in the color accepted for hazardous waste). Disposable bags are placed inside reusable containers. Containers for collecting waste should be marked and installed on a special site (indoor). Reusable containers after emptying should be washed and disinfected. The procedure for washing and disinfecting reusable containers is determined in accordance with the waste management scheme in each specific organization.

Medical devices should be disposed of in accordance with local medical procedures and local, state, and federal regulations. The choice of methods for the safe disposal and destruction of products depends on the capacity and profile of the medical organization, the availability of facilities for the disinfection/disposal of waste, the method of neutralization/destruction of waste adopted in the administrative territory (incineration, removal to landfills, disposal).

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Registration No.219 of 2025

Certificate of Acknowledgment of Notary

On this 8th day of May 2025, TAKAHASHI Ayumi personally appeared before me, KATO Tomohiro, a notary in and for Tokyo Legal Affairs Bureau, with satisfactory evidence of her identification and of her being the agent of Hideki Kazama, President of Tokuyama Dental Corporation, in accordance with the Japanese Notary Act, stated that the principal, said Hideki Kazama, had acknowledged to have affixed his signature to the attached document.

In witness whereof, I set my hand and seal.

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